

## What is the King-Devick Reading Acceleration Program?

The King-Devick Reading Acceleration Program (K-D RAP) is a web-based reading enhancement program that allows patients to practice eye movements that may not be fully developed when a child learns to read.

The patient's task is to read aloud dynamically presented number targets quickly on an iPad or computer. The speed settings are easily adjusted and increased with ongoing practice as the patient improves. K-D RAP has been proven to significantly increase oral reading fluency and comprehension scores in as little as six weeks.



### Patient's K-D RAP Login Details

**Username:** \_\_\_\_\_

**Password:** \_\_\_\_\_

#### Computer Access (Screen Size Minimum: 13")

1. Visit: <https://app.readingacceleration.com>
2. Enter patient's login information above
3. Follow calibration instructions
4. Begin K-D RAP Practice!

#### iPad Access (Screen Size Minimum: 9.7"; iOS 8.0+)

1. Download King-Devick Reading Practice App
2. Launch App and enter patient's login information above
3. Begin K-D RAP Practice!

*Tip: A Wi-Fi connection is required for initial login and to synchronize practice sessions.*

