



## King-Devick Test Online System

King-Devick Test Online System (KDTOS) : Quick Start & Tips.....	2
King-Devick Test Online System Features.....	3
Log into KDTOS.....	5
Calibrate Your Screen.....	5
My Account .....	6
Add New User .....	7
What is a Permission Level?.....	7
Change User Password.....	7
Maintenance .....	7
Import Subjects .....	7
Export Data .....	8
Bulk Delete .....	8
Settings.....	8
Delete Test Records .....	8
Calibrate .....	9
Email Notifications .....	9
Add a Subject .....	9
What are Contact Emails?.....	9
Add a Team .....	9
Administering a Baseline Test.....	9
Administering a Post-Injury Test.....	10
Recording a Test Result.....	11
Troubleshooting.....	12

Your satisfaction is important to us. We're happy to assist you over the phone or via email.  
Please contact us directly at: **+1-844-606-7513** or email us at [support@kingdevicktest.com](mailto:support@kingdevicktest.com).


## King-Devick Test Online System (KDTOS) : Quick Start & Tips

- ✓ The **King-Devick Test Online System (KDTOS)** allows for storage of athletes' K-D Baselines, post-injury screening and monitoring assessments.
- ✓ Login to KDTOS via the web at: [www.kingdevicktestapp.com](http://www.kingdevicktestapp.com)
- ✓ The King-Devick Test has been validated for a standardized test size therefore calibration of the display is required before use. See [Calibrate your screen](#).
- ✓ KDTOS allows you to organize your athletes by team. Quickly and easily import subject details and team rosters to your KDTOS account. See [Import Subjects](#).
- ✓ Use the **King-Devick Test in association with Mayo Clinic** companion tablet app (compatible with 10.1" Android and 9.7" iPad) for testing or when internet is not available. The app synchronizes with your KDTOS account when internet access is available. Visit: <https://kingdevicktest.com/products/concussion/sideline-concussion-screening/resources/> for instructions and tutorials for the *King-Devick Test in association with Mayo Clinic* tablet app.
- ✓ Establish pre-injury K-D Baselines for your athletes. See [Administering a Baseline Test](#) and visit: <https://kingdevicktest.com/products/concussion/sideline-concussion-screening/resources/> for instructions on how to administer a baseline test with a tablet.
- ✓ Screen for injury with K-D Post-Injury Tests. See [Administering a Post-Injury Test](#) and visit: <https://kingdevicktest.com/products/concussion/sideline-concussion-screening/resources/> for instructions on how to administer a post-injury test with a tablet.
- ✓ Additional KDTOS features include:
  - Add Users to your account with a separate login and password
  - Control User access to teams
  - Quickly and easily import subject details and team rosters into your KDTOS account
  - Print subject details including baseline, post-injury assessments and monitoring assessments
  - Export account data and test results
  - Renew or make changes to your KDTOS account

## King-Devick Test Online System Features

### Team Details

The Team Details displays the team details of the selected Team from the left pane Teams list.

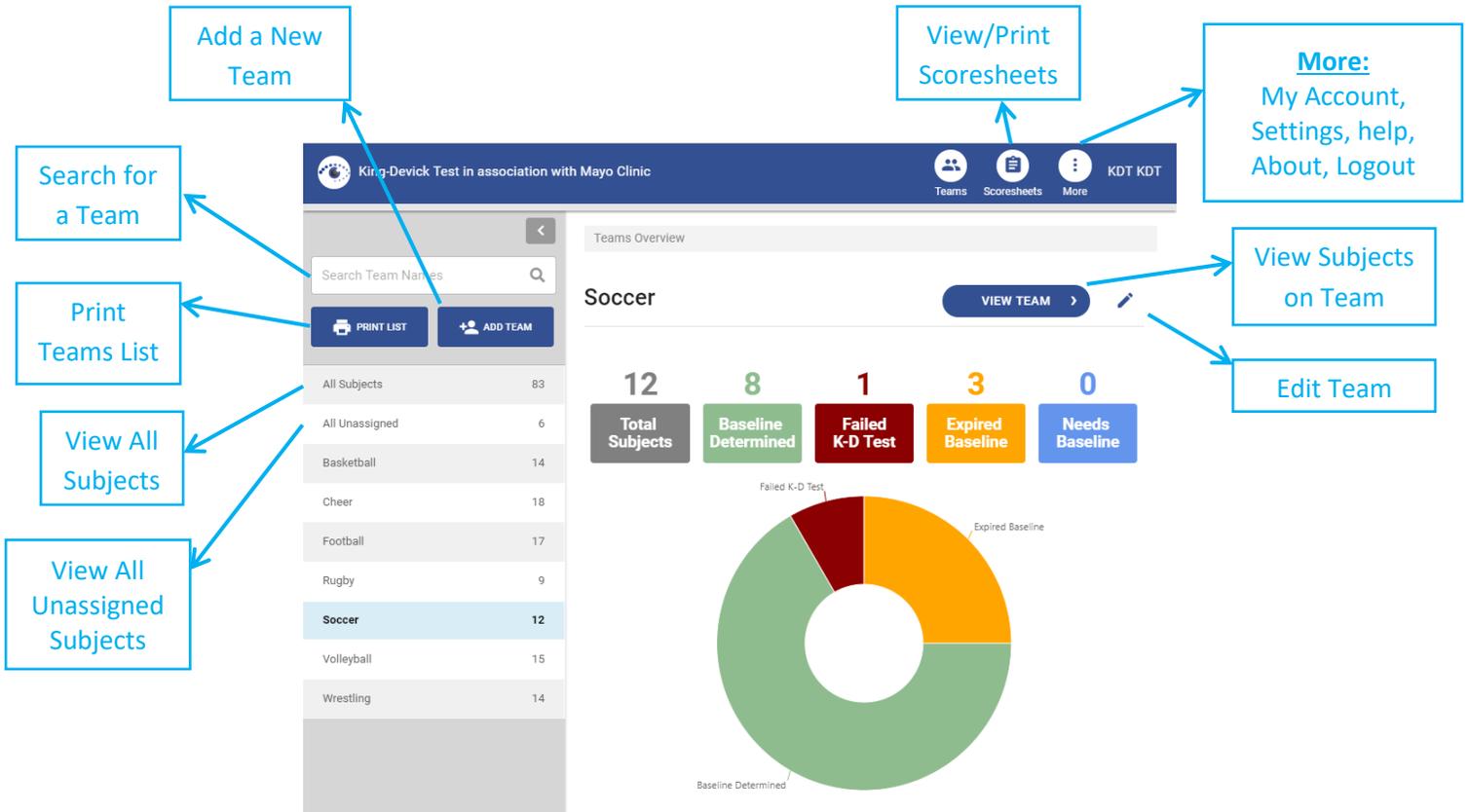

The screenshot shows the 'Team Details' interface for 'Soccer'. The left sidebar contains a 'Teams Overview' section with a search bar and a list of teams. The main content area displays statistics for the selected team and a donut chart showing the distribution of subjects.

**Callout Boxes and Features:**

- Add a New Team:** Points to the 'ADD TEAM' button in the sidebar.
- Search for a Team:** Points to the search bar in the sidebar.
- Print Teams List:** Points to the 'PRINT LIST' button in the sidebar.
- View All Subjects:** Points to the 'All Subjects' row in the sidebar table.
- View All Unassigned Subjects:** Points to the 'All Unassigned' row in the sidebar table.
- View/Print Scoresheets:** Points to the 'Scoresheets' icon in the top navigation bar.
- More: My Account, Settings, help, About, Logout:** Points to the 'More' icon in the top navigation bar.
- View Subjects on Team:** Points to the 'VIEW TEAM' button in the main content area.
- Edit Team:** Points to the edit icon in the main content area.

**Team Statistics (Soccer):**

Total Subjects	12
Baseline Determined	8
Failed K-D Test	1
Expired Baseline	3
Needs Baseline	0

**Donut Chart Data:**

Category	Count
Baseline Determined	8
Expired Baseline	3
Failed K-D Test	1
Needs Baseline	0

# Subject Details

The Subject Details displays the subject details of the selected subject from the left pane subject list.

**Print Subject List**

**Add a New**

**Search for a Subject**

**Sort Subjects by Status**

**View All Teams**

**View/Print Scoresheets**

**More:**  
**My Account:** Add/Edit Users, Import/Export Data  
**Settings:** Delete test records, Calibrate, Email notification settings  
**Help:** Access KDTOS resources  
**About:** KDTOS software details  
**Logout:** Securely log out of your KDTOS account

**Subject Details**

**Cora Alvarado**  
 Subject ID: 26  
 KD-ID: 0611-8201  
 Date of Birth: 30 Mar 2004  
 Status: ✔ BASELINED  
 Expiration: 01 May 2019

**Baseline 49.0s**

**Renew Baseline** **Post-Injury Test**

**Edit Subject Details**

**Print Subject Details**

**Administer a Baseline or Post-Injury Test**

**Record a previous K-D Test**

**View/Edit Subject's Team assignment**

**Add/View Concussion History**

Date	Type	Set	Time	Errors	Cards	Initials	Result
31 May 2018	Post-Injury	1	49s	0	3	AZ	Fail
Comments: Dizzy, headache							
01 May 2018	Baseline		35.2s	0	3	AZ	

## Log into KDTOS

Access the King-Devick Test Online System (KDTOS) via the web at: [www.kingdevicktestapp.com](http://www.kingdevicktestapp.com). Enter your KDTOS User name and Password to access your KDTOS account. If you've forgotten your password, click "Forgot Password".

## Calibrate Your Screen

The King-Devick Test has been validated for a standardized test size therefore calibration of the display is required before use. You can choose to calibrate your screen with either a ruler or a Letter sized (8.5" x 11") sheet of paper.

1. Select to "Use Ruler" or "Use Letter Page".
2. Hold up a ruler or letter page to your computer screen to match the display on the left corner.
3. Drag the displayed ruler or letter page image from the right to match the size of your ruler or letter page.
4. When the horizontal distance of the letter image matches 5 inches on the ruler image matches 5 inches on your ruler or matches your letter page size, click "Done" to save.

Calibration Reset Done

---

You can choose to calibrate your screen with either a ruler or 8.5 inch x 11 inch Letter Page

Use Ruler Use Letter Page

To Calibrate your display:

- Hold a ruler up to the screen.
- Adjust the size of the on-screen red ruler to match 5 inches on your ruler.
- When finished, click on DONE.

If you change the display or display resolution, the system will need to be recalibrated.



Calibration Reset Done

---

You can choose to calibrate your screen with either a ruler or 8.5 inch x 11 inch Letter Page

Use Ruler Use Letter Page

To Calibrate your display:

- Hold a Letter Page up to the screen.
- Adjust the size of the on screen Letter Page to match your Letter Page.
- When finished, click on DONE

If you change the display or display resolution, the system will need to be recalibrated.



## My Account

My Account allows you to view your KDTOS account details. KDTOS Account Administrators may edit KDTOS account details, [Add Users](#), Edit Users and access [Maintenance](#) of your KDTOS Account.

### Maintenance:


**Reports:** View subject status and test summary reports by date

**Import Subjects & Users:** Easily add multiple subjects and additional users to your KDTOS Account

**Export Data:** Download your KDTOS account data

**Bulk Delete:** Quickly delete multiple subjects from your account that are no longer active

Add a New User  
to your KDTOS

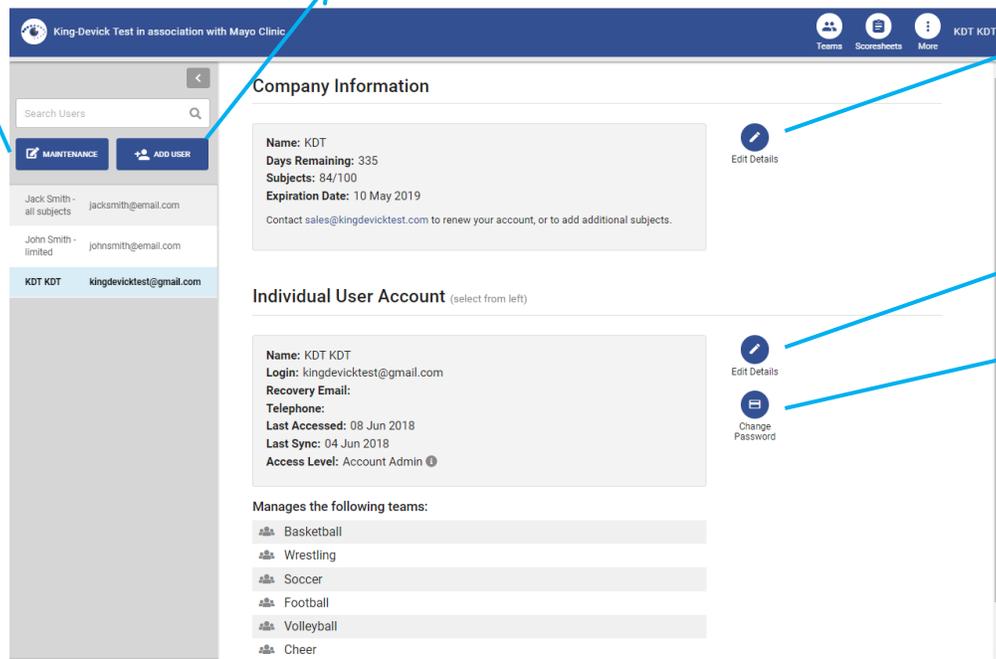

The screenshot shows the 'My Account' page for 'KDT KDT'. The page is divided into several sections:

- Company Information:**
  - Name: KDT
  - Days Remaining: 335
  - Subjects: 84/100
  - Expiration Date: 10 May 2019
  - Contact: sales@kingdevicktest.com
- Individual User Account (select from left):**
  - Name: KDT KDT
  - Login: kingdevicktest@gmail.com
  - Recovery Email:
  - Telephone:
  - Last Accessed: 08 Jun 2018
  - Last Sync: 04 Jun 2018
  - Access Level: Account Admin
- Manages the following teams:**
  - Basketball
  - Wrestling
  - Soccer
  - Football
  - Volleyball
  - Cheer

Callouts from the screenshot include:

- Maintenance:** A large box containing the text from the top section of the document, with an arrow pointing to the 'MAINTENANCE' button in the left sidebar.
- Add a New User to your KDTOS:** A box with an arrow pointing to the 'ADD USER' button in the left sidebar.
- Edit KDTOS account details:** A box with an arrow pointing to the 'Edit Details' icon next to the Company Information section.
- Edit User Details:** A box with an arrow pointing to the 'Edit Details' icon next to the Individual User Account section.
- Change User Password:** A box with an arrow pointing to the 'Change Password' icon next to the Individual User Account section.

## Add New User

KDTOS Administrators may add users to the KDTOS account with separate logins and full or limited visibility and management access of subjects and teams. This “Add New User” function is not available for Team Managers or Testers.

1. Under “My Account”, click “Add User”.
2. Enter the User Profile details: Name, Username, Recovery Email, Phone number (if necessary).
3. Set User Permissions: Select the Permission Level for the new user.

### What is a Permission Level?

The KDTOS allows Account Administrators to give users specific Permission Levels for KDTOS account access. Permission Levels include:

**Account Administrators** have full account access.

**Team Managers** can manage assigned subjects and teams. Select this Permission Level then select which subjects and teams this User will have access to.

**Testers** can administer K-D Test to assigned subjects and teams. Select this Permission Level then select which subjects and teams this User will have access to.

4. Click “Save Changes” to save the new User profile. Users will be sent a Welcome Email once added to your KDTOS account.

## Change User Password

As the account KDTOS administrator you may change User passwords.

**Prior to changing the password be sure to synchronize all tablets associated with that login to ensure that no offline data is lost.**

1. Under “My Account”, select the user from the user list.
2. Click “Change Password”.
3. Prior to changing the password be sure to synchronize all tablets associated with that login to ensure that no offline data is lost. A list of tablets associated with the selected user and the last date that those tablets were synchronized is displayed.
4. If all tablets have been recently synchronized, proceed with changing the password. Enter the new password in the corresponding fields. Click “Save & Exit”.
5. All tablets associated with that user login account will need to log out and log in with the newly established password to continue to synchronize correctly.

## Maintenance

Maintenance of your KDTOS Account allows you to [Import Subjects](#), [Export Data](#), [Bulk Delete](#).

### Import Subjects

Quickly and easily import subject details and team rosters into your KDTOS account.

1. Under “My Account”, click “Maintenance” then “Import Subjects”
2. Click to download the Subject Import Template.
3. Enter your account subjects into the template.
  - a. Required fields include: First Name, Last Name, and Date of Birth.
  - b. Date of Birth should be entered in MM/DD/YYYY format.
  - c. Contact email allows post-injury K-D Test notifications to be sent if the feature is enabled. See [Email Notifications](#).

- d. Baseline and Baseline Date: if available a previously established K-D Baseline may be imported with a Baseline Date.
  - e. Teams: assign subject to a team or multiple teams. List additional teams in adjacent columns.
4. Save your complete Subject Import File as a CSV file to your computer.
  5. Click “Next”.
  6. Click “Choose File” to locate your saved Subject Import File.
  7. Click “Next”. The KDTOS will check your data file and report any errors. Follow instructions to complete the import process.

## Export Data

Export Data allows you to quickly download and export your current account data.

1. Under “My Account”, click “Maintenance” then “Export”.
2. Select which type of export:
  - Summary Export** provides Current Subject Status and Test Summary
  - Details Export** provides a summary report and individual subject detail reports.
  - Single Sheet Export** provides a single sheet summary of individual subjects organized by row and test administration.

The export will automatically begin downloading as an Excel file.

## Bulk Delete

Bulk Delete allows you to quickly and easily remove subjects you no longer want to include in your account (i.e. student-athletes that have graduated).

1. Under “My Account”, click “Maintenance” then “Bulk Delete”.
2. Click the column headers to sort by column, drag columns above the headers to group by columns.
3. Select the subjects to be deleted. Highlight multiple subjects by using SHIFT or CTRL key.
4. Click “Delete Subject” to bulk delete the selected subjects.

## Settings

The KDTOS Settings allows Administrators to Transfer Subjects between teams, [Delete Test Records](#), [Calibrate](#) and set [Email Notification](#) preferences.

## Delete Test Records

Manually remove incorrect test records.

1. Turn this feature ON here in Settings. The feature will remain ON for your current login session and you may view all delete history by clicking “View Logs”
2. Return to the Subject Profile that you would like to make a change.
3. Under K-D Test History, find the test record that you would like to remove.
4. Click the trash icon to delete the individual test record. The subject’s status will update to reflect the new K-D Test History.

## Calibrate

Re-calibrate your screen. The King-Devick Test has been validated for a standardized test size therefore calibration of the display is required before use. You can choose to calibrate your screen with either a ruler or a Letter sized (8.5" x 11") sheet of paper. See [Calibrate Your Screen](#).

## Email Notifications

Expiring Baselines: Enable monthly administrator account email notifications of expiring baselines.

Post-Injury Emails: Send Subject contact email notification of post-injury tests.

## Add a Subject

Add a New Subject's details to your account. This function is available to KDTOS account Administrators and Team Managers.

1. Click "Teams" from the top toolbar.
2. Select the team that you would like to add the new subject to. Click "View Team".
3. Click "Add Subject" and enter the subject's details including required fields: Name and Date of Birth. Subject ID and Contact Emails are optional.
4. Click "Save Changes" to save the new subject profile. Your new subject now appears in the subject list. See [Administering a Baseline Test](#).

## What are Contact Emails?

Enter contact emails to send notifications of post-injury tests. This setting can be turned on and off by the Administrator under KDTOS Account Settings. See [Email Notifications](#).

## Add a Team

The KDTOS allows you to create Teams to help organize your subjects. This function is available to KDTOS account Administrators and Team Managers with access to all subjects.

1. Click "Teams" from the top toolbar.
2. Click "Add Team".
3. Enter the Team Name.
4. Add subjects to the new team by selecting from the subject list. You may also search for subjects by name or filter subjects by existing teams.
5. Click "Save Changes" when complete. Your new team now appears in your team list.

## Administering a Baseline Test

KDTOS allows you to administer a Baseline Test directly on a calibrated computer. Be sure to [calibrate your screen](#) before testing. A **Baseline Test** is a pre-injury baseline test score determined by following the Baseline Testing protocol which establishes a baseline test score as the faster of two error free trials. Establish a K-D Baseline for new subjects or Renew a Baseline for subjects with a previous K-D Baseline.

1. Select the subject from the subject list that you would like to test.
2. The test administrator should have the corresponding Scoresheet ready to tally errors. Download or email Scoresheets by clicking on "Scoresheets" from the top toolbar or use the mobile Scoresheet at: <https://www.kingdevicktestapp.com/phone>.
3. Click "Baseline Test" for new subjects or "Renew baseline" for subjects with a previous K-D Baseline.
4. Read the instructions carefully. Select the Test Set.

## What is the Test Set?

There are 3 test sets available. Test sets are the same King-Devick task with different number sets to minimize memorization. As the test administrator, be sure to have the corresponding Test Set Score Sheet to follow along for errors.

5. Click "Start Test" to begin the test.
6. The Demonstration Card is displayed, and no active, automatic timing is occurring. The test administrator should explain the task to the subject. When both the test administrator and subject are ready to begin, the test administrator should click the screen to control the automatic timing and advance to Test Card 1. The subject should immediately begin reading aloud the numbers and the test administrator should follow along on their Scoresheet to mark any errors.
7. When the subject calls out the last number on Test Card 1, the test administrator should click on the screen to pause timing. A break card is displayed and no active, automatic timing is occurring. When both the test administrator and subject are ready to continue, the test administrator should click anywhere on the screen to control the automatic timing and advance to Test Card 2. Repeat this for Test Card 3.
8. On the Test Results page, enter the Number of Errors, the Tester Initials and any comments. Click "Confirm" to save the result or click "Cancel" to cancel and discard this result.
9. Baseline testing requires two error free test trials. Follow the on screen prompts to complete Baseline testing. Click "Save Test" to save the result in the subject's K-D Test History and continue testing.
10. When two acceptable baseline trials have been administered, you will receive a notice that the Baseline has been determined. The Baseline test result is now saved in the subject's K-D Test History.

### TROUBLESHOOTING:

- Baseline testing requires two test administrations. If you've run one baseline test but the subject still has a YELLOW status indicator, please administer another baseline test until you reach the results page that displays "Baseline Determined. Thank you."
- Test Set: There are 3 test sets available. Test sets are the same King-Devick task with different number sets to minimize memorization. As the test administrator, be sure to have the corresponding Test Set Score Sheet to follow along for errors.

## Administering a Post-Injury Test

The KDTOS allows you to administer a Post-Injury Test directly on a calibrated computer. Be sure to calibrate your screen before testing. A **Post-Injury Test** is any test administration following suspected head trauma to screen for injury and/or following confirmed head trauma to monitor recovery.

1. Select the subject from the subject list that you would like to test.
2. The test administrator should have the corresponding Scoresheet ready to tally errors. Download or email Scoresheets by clicking on "Scoresheets" from the top toolbar or use the mobile Scoresheet at: <https://www.kingdevicktestapp.com/phone>.
3. Click "Post-Injury Test".
4. Read the instructions carefully. Select the Test Set.

### What is a Test Set?

There are 3 Test Sets available through KDTOS. Test Sets are the same King-Devick task with different number sets to minimize memorization. As the test administrator be sure to have the corresponding Test Set Scoresheet to follow along for errors.

5. Click "Start Test" to begin the test.

6. The Demonstration Card is displayed and no active, automatic timing is occurring. The test administrator should explain the task to the subject. When both the test administrator and subject are ready to begin, the test administrator should click anywhere on the screen to control the automatic timing and advance to Test Card 1. The subject should immediately begin reading aloud the numbers and the test administrator should follow along on their Scoresheet to mark any errors.
7. When the subject calls out the last number on Test Card 1, the test administrator should click on the screen to pause timing. A break card is displayed and no active, automatic timing is occurring. When both the test administrator and subject are ready to continue, the test administrator should click anywhere on the screen to control the automatic timing and advance to Test Card 2. Repeat this for Test Card 3.
8. On the Test Results page, enter the Number of Errors, the Tester Initials and any comments. Click "Confirm" to save the result or tap "Cancel" to cancel and discard this result.
9. The test result is automatically compared to the individual subject's pre-injury Baseline and results are displayed. Tap "Save Test" to save this result.
10. The test result is now saved in the subject's K-D Test History.

## **Recording a Test Result**

The KDTOS allows you to manually enter and save previous test results.

1. Select the subject from the subject list that you would like to test.
2. Under the K-D Test History, click "Record Test".
3. In the Record Result page, select which type of test you are recording.

### **What is a Baseline Test?**

A Baseline Test is a pre-injury baseline test score determined by following the Baseline Testing protocol which establishes a baseline test score as the faster of two error free trials.

Enter in the final Baseline Time (in seconds), Test Date, Number of Test Cards used, Tester Initials, and any additional comments. Click "Continue" then "Save Test" to save this result in the subject's K-D Test History.

### **What is a Post-Injury Test?**

A Post-Injury Test is any test administration following suspected head trauma to screen for injury and/or following confirmed head trauma to monitor recovery.

Record the total Time (in seconds), Test Date, Number of Errors, and Tester Initials. In the comments you may add observations about the suspected injury (i.e. how the injury was sustained, acute signs and symptoms exhibited, etc.). Click "Continue" then "Save Test" to save this result in the subject's K-D Test History.

## **Troubleshooting**

Your satisfaction is important to us. We're happy to assist you over the phone or via email. Please contact us directly at: **+1-844-606-7513** or email us at [support@kingdevicktest.com](mailto:support@kingdevicktest.com).

### **Can't log in**

If you receive this error message during the KDTOS log in:

1. Please check your Username and Password. The Password entry is case sensitive.
2. Be sure that you have a working internet connection.

## Contact Details

### Corporate Headquarters

King-Devick technologies, inc.  
2 Mid America Plaza, Suite 110 Oakbrook Terrace,  
IL 60181,  
USA

Telephone: +1 844 606 7513  
Email: [support@kingdevicktest.com](mailto:support@kingdevicktest.com)

### European Authorized Representative

Waterfall Consultants trading as RPC Consultants,  
Ballyleigh,  
Waterfall,  
Co Cork,  
Ireland

Telephone: +353 21 239 4662  
Email: [kdt@rpcconsultants.com](mailto:kdt@rpcconsultants.com)

### Australian Authorized Representative

Emergo Australia  
Level 20, Tower II  
Darling Park  
201 Sussex Street  
Sydney, NSW 2000  
Australia